

QUICKSTART

- 1) Position the patient
- 2) Move the EPD into place and lock the casters.
- 3) Adjust the EPD to the proper height
- 4) Have the patient lean forward. Adjust the Face Cradle.
- 5) Adjust the Arm Rests.
- 6) Adjust the Foot Rests.
- 7) Loosen the knob to push the chest cushion into the chest - *(making a nice curve in the back).*
- 8) Perform the Epidural Procedure
- 9) Help the patient lean back -off the machine
- 10) Unlock the casters, move the EPD out of the way.

